






EasyGuide™: ACL Graft Prep Examples

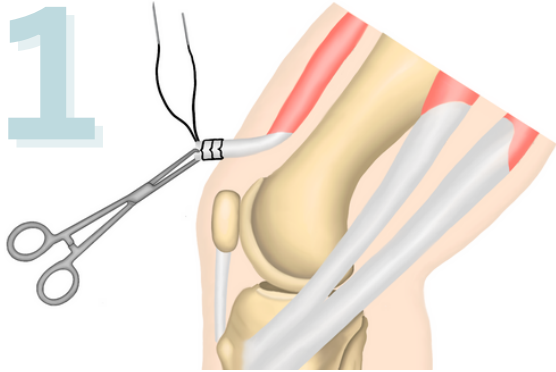


1
Stitch the tendon at the incision for autograft harvesting

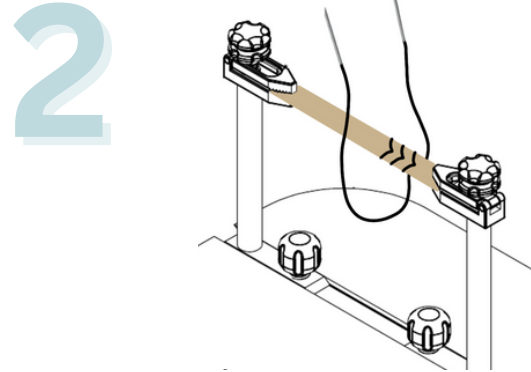


2
Stitch on a prep stand with **both** graft ends secured to the clamps

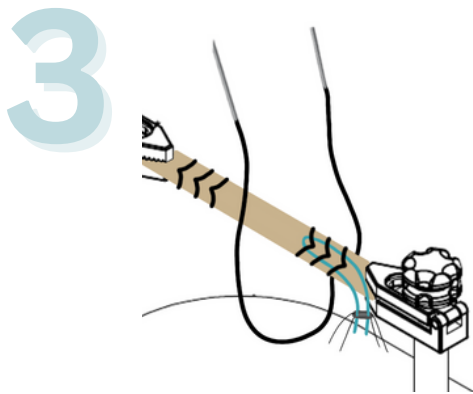

3
Attach a suspensory button to the graft

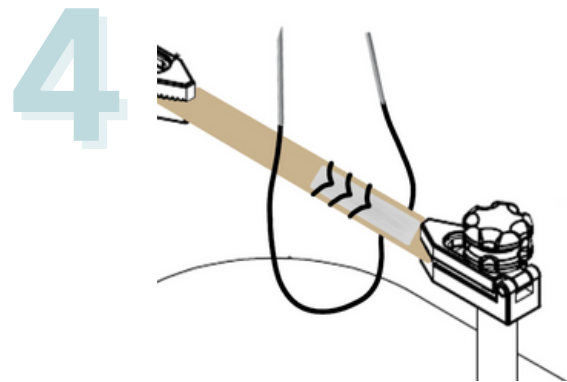

4
Attach an augmentation product to the graft as you normally would

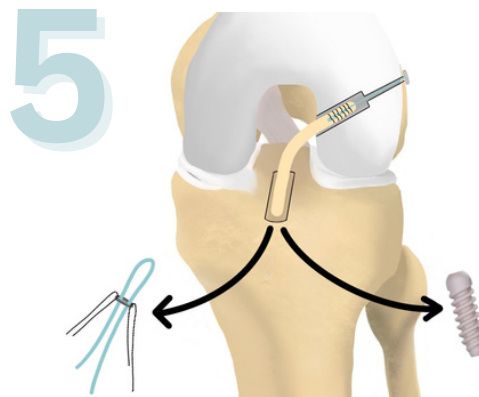

5
Use the suture tails to implant and tension and fix the stitched graft in the knee



www.winter-innovations.com
info@winter-innovations.com

©2024 Winter Innovations.
All rights reserved.
MKT508A 2024-3

This surgical technique is provided to aid clinicians using EasyWhip® for standard of care procedures. EasyWhip® is indicated for use in approximation and/or ligation of soft tissues, including the use of allograft tissue for orthopedic surgeries. EasyWhip® simplifies and standardizes the same suture ties that can be performed manually. *Reference:* Prodrmos, C. The Anterior Cruciate Ligament: Reconstruction and Basic Science E-Book. (Elsevier Health Sciences, 2017). Other augmentation and fixation products mentioned in this guide are routinely needed and used in conjunction with EasyWhip®.

Content is intended for educational purposes only. Surgeons should rely on their professional expertise, review product guidelines, and be trained on products before use in surgery. Refer to the Instructions For Use for full prescribing information www.winter-innovations.com/eifu EasyWhip® is protected under granted and pending US and international patents. More at www.winter-innovations.com/patents