
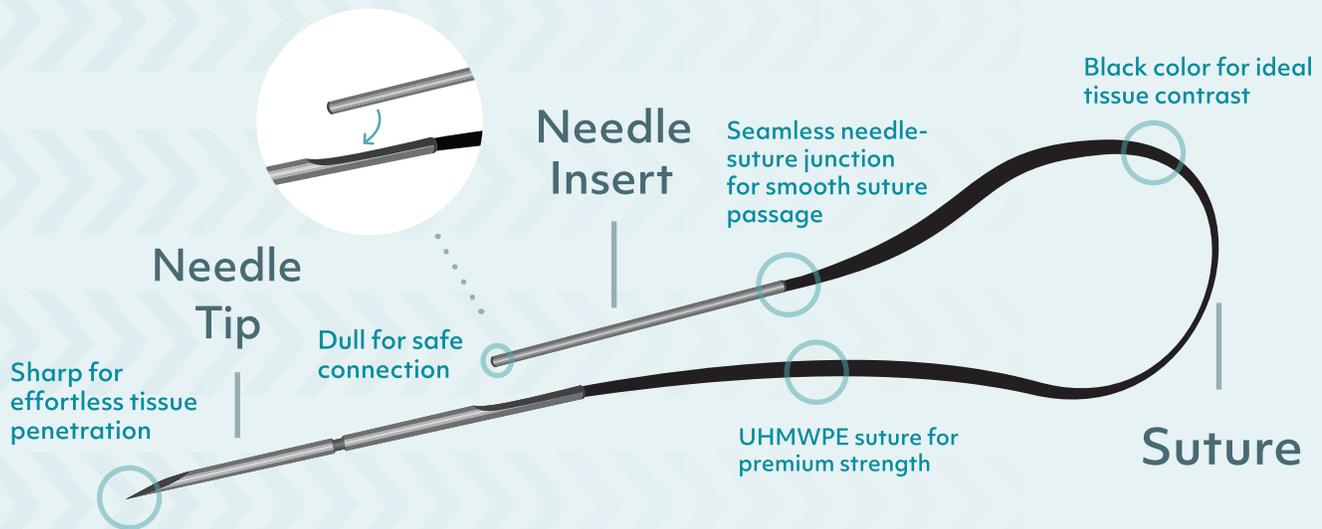


Easy Whip™

surgical suturing needle

The future for suture


Introducing EasyWhip™ an innovative suturing needle that makes stitching **FAST, SECURE, and ACCURATE.**
In a word, **EASY!**

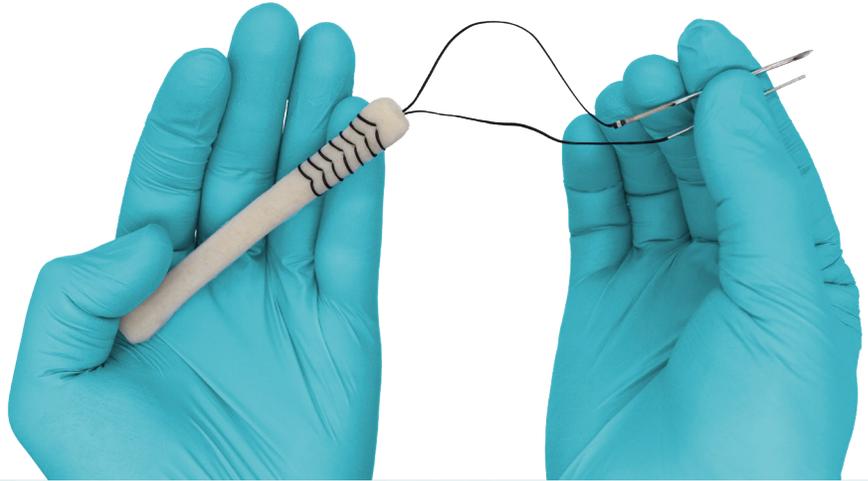

The novel two-part needle design allows for case-by-case flexibility that is not possible with conventional suturing needles.

Choose from multiple stitch techniques or create custom patterns based on the needs of each case.

* The WhipLock™ is a proprietary locking stitch that is a cross between a whip stitch and a Krackow stitch

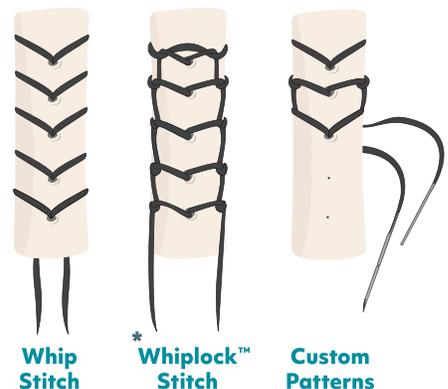

Comparison testing on porcine tendon demonstrates benefits of EasyWhip™

Results based on pre-clinical bench testing performed on ex vivo porcine tissue, data on file. Clinical results in humans are unknown.


EasyWhip™

Conventional Needle

| Fast


The two-part needle design of EasyWhip™ saves time by avoiding extra movements required by conventional looped and curved needles. EasyWhip™ demonstrated time savings compared to both whip and locking stitches.

WHIP STITCH

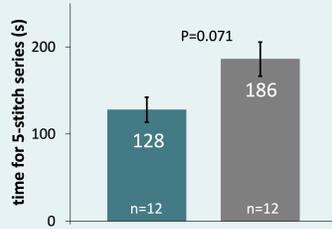

30% less time for stitching

LOCKING STITCH

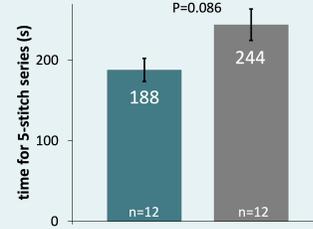


25% less time for stitching

| Secure


EasyWhip™ was shown to create secure constructs with demonstrated advantages in strength, stability, & reliability. Results below compare bench testing of ex vivo porcine tendon **whip stitched** using EasyWhip™ vs. a conventional needle with loop suture.

STRONG

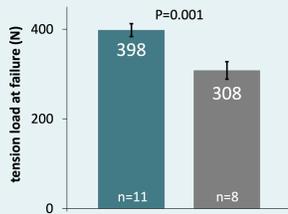


29% higher failure load

STABLE

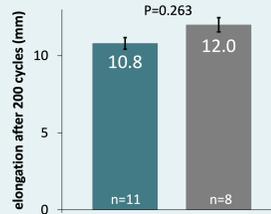

10% less elongation

RELIABLE

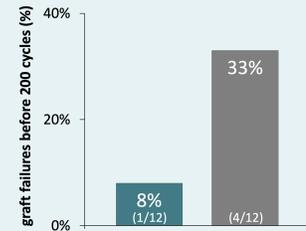


3X fewer premature failures

| Accurate

EasyWhip™ allows the graft to be securely fixed on both ends during stitching. This is convenient, as it allows one person to perform the procedure, and it enables accurate stitch placement with less variability.


Whip Stitching with Conventional Needle

*graft is securely fixed on one end

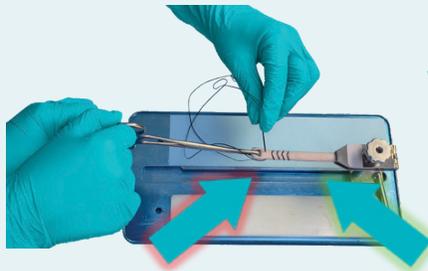

Whip Stitching with EasyWhip™

*graft is securely fixed on both ends

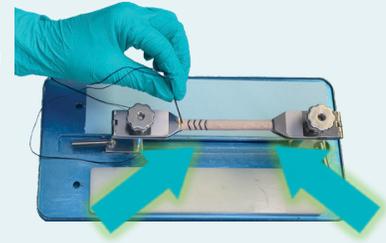

Product Information



Needle

Material: Stainless steel
Configuration: 2-part needle

Suture

Size: #2 (Metric 5)
Color: Black
Configuration: Braided, coreless

Reference

EW-01-1: EasyWhip™ Individual Pack, 1 unit
EW-01-10: EasyWhip™ Multi-Pack, 10 units

Indications

EasyWhip™ is indicated for use in approximation and/or ligation of soft tissues, including the use of allograft tissue for orthopedic surgeries.

510(k) K210675

Refer to the instructions for use for information on indications for use, contraindications, potential complications, warnings, and precautions (winter-innovations.com/eifu)